

EXPLANATION OF SYMBOLS USED ON THIS DEVICE

Not all features included with each model.

FUNCTION	SYMBOL	EXPLANATION
POWER (SB)		Turns unit On / Off. Green = On Orange = Standby
SOFT / FIRM (K)		Up or Down key adjusts patient comfort pressure levels.
PLUS / MINUS (L)		Adjust "+/-" to set various therapy times and patient settings.
SET-UP (H)		Set Patients height and weight.
MODE (M)		Select between Therapy Modes. (K-3, K-4, K-4oem)
SELECT (S)		Select between Therapy Modes. (K-3oem)
ALARM SILENCE (AS)		Alarm Silence – mutes audible alarm.
LOCK OUT (LO)		Locks out all control unit functions to prevent tampering. (Max Inflate still available on K-3 & K-4)
PULSE (N) (WAVE)		Selects Pulsation mode on. Continuous Low Air Loss during Pulse mode.
PAN (Y)		Selects Pan mode on. Continuous Low Air Loss during Pan mode.
STATIC (T)		Selects Static Therapy mode. Continuous Low Air Loss during Static mode.
ALTERNATING PRESSURE (AP)		Selects A/P mode. Continuous Low Air Loss during Pulse mode.
10 MIN A/P (D)		Default AP -Preset 10 min Alternating Pressure Therapy. Continuous Low Air Loss during A/P mode.
FOWLER (U)		Fowler Mode. The pressure will increase to prevent the patient from bottoming out.
MAX Inflate (W)		Inflates mattress rapidly (15 minute timer) Continuous Low Air Loss during A/P mode.
POWER FAIL (PF)		In the event of power failure or if the hose is disconnected, an audio/visual alarm will sound.
LOW PRESSURE (LP)		
	Indicates the point of attachment of the equipment to earth (Grounding Point).	
	Attention: Instructs end user / care giver / operator to refer to the manual.	
	Indicates that the degree of protection against electrical shock is TYPE BF.	
	Not for use in presence of flammable anesthetics.	
	Consult Instructions for Use	
	Waste electrical and electronic equipment (recycle).	
	Risk of electrical shock. Do not remove back cover.	