

Citrus II®

HOSPITAL GERMICIDAL DEODORIZING CLEANER

EFFICACY TESTS HAVE DEMONSTRATED THAT THIS PRODUCT IS AN EFFECTIVE BACTERICIDE, VIRUCIDE AND FUNGICIDE

(SEE BACK PAGE FOR KILL TIMES AND EFFICACY DATA EPA REGISTERED)

Ideal for:

- Hospitals
- Doctors' Offices
- Nursing Homes
- Surgery Centers
- Home Health Care Equipment Dealers
- Labs
- Blood Banks
- Daycare Centers
- Patients' Home Care
- Flood Clean-Up

Advantages of Citrus II

Hospital Germicidal Cleaner:

- Meets OSHA Bloodborne Pathogen Standard for HIV, HBC, HCV & HAV
- Ready-to-Use Formula
- Non-Alcohol Formula
- Non-Flammable
- Will not dry out vinyl or Naugahyde
- Less expensive to use
- Multiple Use Formula
- Cleans without scrubbing
- Wipes clean without leaving grit or soap scum
- Eliminates odors caused by micro-organisms
- Broad Spectrum Antimicrobial Activity
- Fresh citrus fragrance

Safe on Surfaces - Citrus II is ideal for cleaning and disinfecting :

- Countertops
- Vinyl and Naugahyde Surfaces
- Waiting and Exam Rooms
- Bed Springs
- Telephones
- Wheelchairs
- Patient Rooms
- Sinks
- Bathrooms
- Shower Stalls
- Plastic
- Chrome
- Stainless Steel
- Plexiglas, Walls
- And Other Non-Porous Surfaces

Note: As is the case with all cleaning products, test on an inconspicuous area for color fastness.

USDA APPROVED!

- Kills Salmonella Choleraesuis
- Ideal for use in kitchens and food preparation areas
- May be used in kitchens to clean, disinfect and deodorize counters, sinks, appliances, and stove tops.

Note: A rinse with potable water is required for surfaces in direct contact with food.

Non-Critical Instruments

Citrus II may be used to clean and decontaminate non-critical instruments and devices that come into contact with intact skin.



Beaumont Products, Inc. • Kennesaw, GA 30144
1-800-451-7096 • Fax 770-514-7400 • www.citrus2.com



Citrus II®

HOSPITAL GERMICIDAL DEODORIZING CLEANER

CITRUS II HOSPITAL GERMICIDAL DEODORIZING CLEANER is designed specifically as a general non-acid ready-to-use cleaner and disinfectant.

- EFFICACY TESTS HAVE DEMONSTRATED THAT THIS PRODUCT IS AN EFFECTIVE BACTERICIDE, VIRUCIDE AND FUNGICIDE.
- It is formulated to disinfect hard, non-porous, inanimate environmental surfaces, equipment and non-critical instruments.
- In addition, this product deodorizes those areas which generally are hard to keep fresh-smelling.

BACTERICIDAL ACTIVITY :

Citrus II kills the following organisms in 3 minutes:

- Pseudomonas aeruginosa, (ATCC 15442)
- Staphylococcus aureus, (ATCC 6538)
- Salmonella choleraesuis (ATCC 10708)
- Escherichia coli, (ATCC 11229)
- Escherichia coli, 0157:H7, (ATCC 43895)
- Listeria monocytogenes, (ATCC 35152)
- Yersinia enterocolitica, (ATCC 23715)
- Enterococcus faecium, (ATCC 6569)
- Corynebacterium ammoniagenes, (ATCC 6871)
- Salmonella typhi, (ATCC 6539)

STRAINS OF ANTIBIOTIC-RESISTANT BACTERIA

- Methicillin resistant Staphylococcus (MRSA), (ATCC 33593)
- Vancomycin resistant Enterococcus faecalis (VRE), (ATCC 51575)
- Vancomycin intermediate resistant Staphylococcus aureus (VISA), (CDC Isolate 99287)

TUBERCULOCIDAL ACTIVITY:

- Kills Mycobacterium Tuberculosis in **5 minutes**.

***VIRUCIDAL ACTIVITY:**

- **Kills the following viruses in 10 minutes:**
 - HIV-1 (associated with AIDS)
 - Hepatitis A Virus (HAV)
 - Hepatitis B Virus (HBV)
 - Hepatitis C Virus (HCV)
 - Poliovirus Type 1
 - Canine Parvovirus
 - Bovine Viral Diarrhea Virus (BVDV)
- **Kills the following viruses in 2 minutes.**
 - Norwalk Virus
 - Human Coronavirus
 - Feline Calicivirus
- Kills Rabies Virus in **30 seconds**.

FUNGICIDAL ACTIVITY:

Kills Pathogenic Fungus, Trichophyton Mentagrophytes in **10 minutes**.

MILDEWSTAT:

To control mold and mildew on pre-cleaned, hard, non-porous surfaces, spray surface to be treated making sure to wet completely. Let air dry. Repeat application at weekly intervals or when mildew growth appears.