

RescueNet Code Review - all editions

Hardware and software requirements

Document updated: March 10, 2004

- RescueNet Code Review - all editions 1
 - Computer 1
 - Operating system 1
 - Internet 1
 - Tested infrared PC ports and adapters 1
 - Tested card readers 2
- ZOLL InterLink 2
 - Pocket PC 2
 - Tested Pocket PC devices 2
- Prerequisites for transmitting real-time 12-lead / vital trends 3
 - Adobe Acrobat Reader 3
 - Symantec WinFax PRO 9.0 or higher 3
 - LEADTOOLS ePrint 3

RescueNet Code Review - all editions

Computer

- Pentium II microprocessor or higher
- 256 MB RAM
- 50 MB free hard disk space
- CD drive or internet connection
- Optional Infrared PC port or adapter for the data download from AED Plus (see the following *Tested infrared PC ports and adapters* section)
- Optional PC card reader (Type 2 Linear flash) for the card download from M Series (see the following *Tested card readers* section)

Operating system

- Microsoft Windows 2000 or XP Professional (recommended)
- Microsoft Windows NT 4.0 (service pack 5 or later)
- Microsoft Windows ME
- Microsoft Windows 98

Internet

Optional: Internet connection for enhanced access to support services.

Tested infrared PC ports and adapters

- Most laptops with integrated infrared port.
- USB infrared adapter, available from ZOLL Medical Corporation, part number 8000-0815.
- RS232 (serial) infrared adapter (recommended for use with Microsoft Windows 98 operating system), available from ZOLL Medical Corporation, part number 8000-0816.

Tested card readers

- **Most** laptops with integrated type 2 PC card slot.
- OmniDrive Pro external card reader (Parallel). Available from ZOLL Medical Corporation (P/N 8000-0692-01), or directly from www.Intrpducts.com.

ZOLL InterLink

Pocket PC

- Pocket PC device running Pocket PC 2002 or higher
- 32 MB RAM/32 MB ROM (Flash)
- Cradle
- Color or grey scale touch screen

Tested Pocket PC devices

- Compaq iPAQ 3870
- Compaq iPAQ 3975
- T-Mobile Pocket PC Phone

For more information about Pocket PCs, visit www.PocketPCCentral.net.

Prerequisites for transmitting real-time 12-lead / vital trends

The workstation must have a WinFax PRO-compatible fax modem, and you must install the following three 3rd-party tools on the workstation:

Adobe Acrobat Reader

A free download is available at www.adobe.com/products/acrobat/readstep2.html.

Symantec WinFax PRO 9.0 or higher

You can purchase WinFax PRO at www.symantec.com/winfax/.

LEADTOOLS ePrint

Intercepts printer output and generates various document formats such as PDF and DOC (Word). You can purchase ePrint at www.eprintdriver.com.

When you install this utility, the “LEADTOOLS ePrint” printer is added to your system.

Important! The ePrint printer must be the default printer, and its name must not be changed from “LEADTOOLS ePrint.”

Before transmitting reports, you must configure the ePrint printer:

1. Open the **Properties** dialog box for the LEADTOOLS ePrint printer:
On the **Start** menu, point to **Settings**, and then click **Control Panel**. In the **Control Panel** dialog box, double-click **Printers and Faxes**. In the **Printers and Faxes** dialog box, right-click LEADTOOLS ePrint, and then click **Properties**.)
2. Move to the Advanced tab and select Print directly to this printer.
3. Move to the **Save** tab and select **Use save options** with these settings:
Directory. Click Browse and select the Images\Temp folder for ZOLL Data (typically C:\Program Files\Pinpoint Technologies, Inc\ZDR\Images\Temp).

The **Use document** name check box must be cleared.

Although any document type supported by WinFax PRO is compatible, it is recommended that you use the **PDF document type**. Use these settings:

- File name = image.pdf
- File type = document file

- Save as type = PDF
 - Multipage = Each print as a separate file
4. Move to the **Options** tab and configure these settings:
- In the **Usage** section:
- Select Use save settings.
 - Clear the Use email settings and Use printer settings check boxes.
- In the **General** section:
- Select Show before each print and Show status dialog.