LIMITED WARRANTY

What does the Warranty Cover?

Pelstar LLC scales are warranted from date of purchase against defects of materials or in workmanship for a period of one (1) year. If product fails to function properly, return the product, prepaid and properly packed to Pelstar. If manufacturer determines that a defect of material or in workmanship exists, customers' sole remedy will be repair or replacement of scale at no charge. Replacement will be made with a new or remanufactured product or component. If the product is no longer available, replacement may be made with a similar product of equal or greater value. All parts including repaired and replaced parts are covered only for the original warranty period.

Who is Covered?

The original purchaser of the product must have proof of purchase to receive warranty service. Pelstar dealers or retail stores selling Pelstar products do not have the right to alter, or modify or any way change the terms and conditions of this warranty.

What is Excluded?

Your warranty does not cover normal wear of parts or damage resulting from any of the following: negligent use or misuse of the product, use on improper voltage or current, use contrary to the operating instruction, abuse including tampering, damage in transit, or unauthorized repair or alternations. Further, the warranty does not cover Acts of God, such as fire, flood, hurricanes and tornadoes. This warranty gives you specific legal rights, and you may also have other rights that vary from country to country, state to state, province to province or jurisdiction to jurisdiction.

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Contactez le Service Garanties au 1-800-815-6615.

Another innovation from the people at

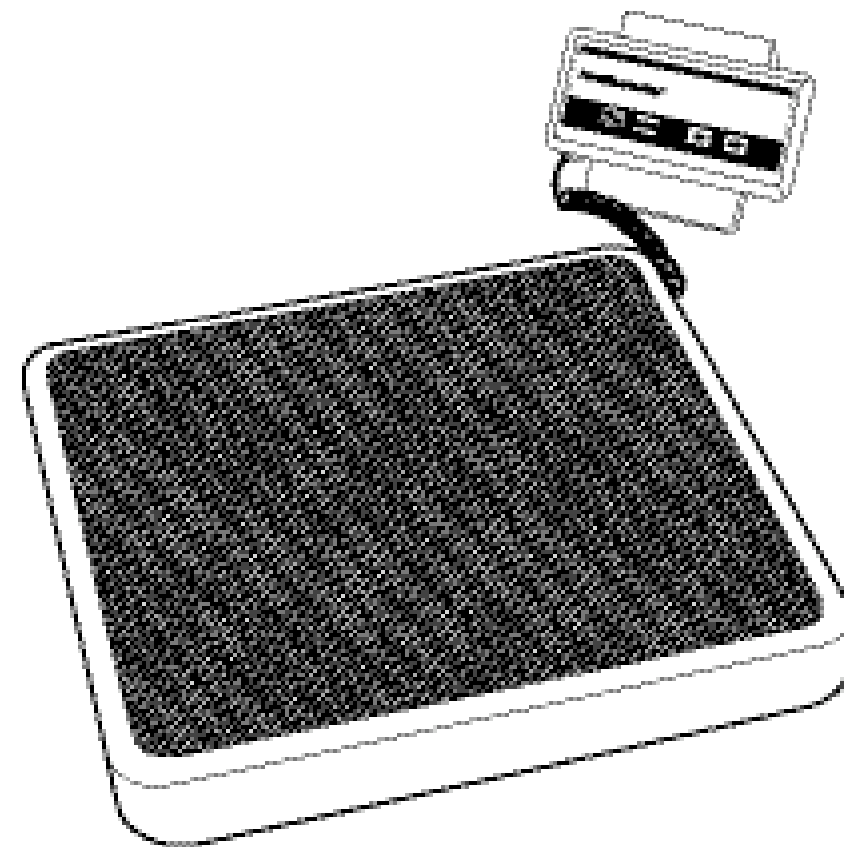
Health o meter
Professional

7400 West 100th Place, Bridgeview IL 60455 • 1-800-815-6615
www.healthometermedical.com

Health o meter® Professional products are manufactured, designed and owned by Pelstar LLC.
Health o meter® is a registered trademark of Sunbeam Products Inc., Boca Raton, FL 33431

Health o meter[®]
Professional

OPERATING INSTRUCTIONS INSTRUCCIONES DE OPERACIÓN MODE D'EMPLOI

PHYSICIANS REMOTE READ DIGITAL SCALE

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MODÈLE: 320KL / 349KLX

PHYSICIANS REMOTE READ DIGITAL SCALE

MODELS: 320KL / 349KL

USING THE SCALE

LB/KG: Pressing this button will change the unit of measure displayed. This scale will weigh in pounds (LB) or in kilograms (KG).**

ZERO: Pressing this button will reset the scale display to zero: 0.0.

Preparing Scale for Use

1. Remove the plastic film from display.
2. Plug the coiled, "phone" cord into the bottom left of the remote head.
3. Place scale on the floor on a vibration-free, level surface.
4. Press "ON". Wait for the display to show Zero in the units of measure last used (to change the units of measure, see section "Using the Scale: LB/KG" below). You are now ready to begin weighing.
5. Always allow the display to return to Zero before stepping on the platform.

Special Features

AUTO SHUT-OFF: Scale will automatically turn off after approximately 2 minutes of inactivity.

DEACTIVATING AUTO SHUT-OFF: While scale is off, press and hold the "LB/KG" button. While still holding "LB/KG" press "ON". "A_of" will display. The scale will remain on until manually turned off.

Understanding Messages

"oooo": the weight capacity has been exceeded
"-3.0": minus weight, to adjust press ZERO
"bAtt": Blinking means the battery is low

POWER SUPPLY

Your new scale operates on one 9V battery or 110V AC adapter output: 9V DC 100mA (included).

BATTERY USE: Battery compartment is located on the underside of the remote display head. Connect battery ± accordingly.

ADAPTER USE: First plug into bottom right of remote head, then plug into wall outlet.

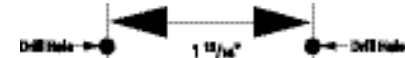
NOTE: When using the AC adapter, if display fluctuates between "-----" and numbers, reverse adapter in wall outlet.

REMOTE HEAD MOUNTING

The remote head is attached to the scale by a coiled "phone" cord. The remote head can be mounted to a wall up to 9 feet from the base, or it can be mounted on the display bracket (included) for placement on a table up to 9 feet from the base.

To Mount on Wall

1. Use the above diagram to locate the position of the screws at the desired height.


2. Drill two holes 1 13/16" apart.

3. Screw the two screws into the wall, let the heads protrude 1/8" from the wall. Extending the screw heads further could damage the scale.

4. Align the holes on the back of the remote head with the screws. The remote head should slide onto the screw heads, firmly and securely.

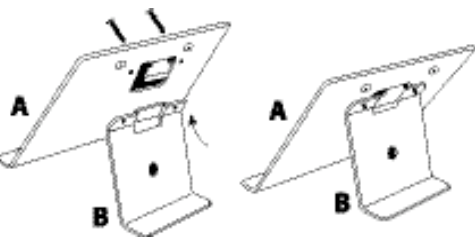To Mount on Display Bracket

1. Hold Part A (Stand) with the knobs facing away from you. Slide the center tab of Part B (Support) up and under the tab on the back of the Stand. Be sure to align the two parts, so that the tab on the Support can slide beneath the tab on the rear of the Stand. The holes on both parts of the bracket should now line up.

2. Turn the bracket over so Part A is facing you and insert the two screws (included) into the holes and through the openings in Part B.

3. Using a Phillips head screwdriver, tighten the screws to secure Part B to Part A.

4. Align the holes in the back of the remote display head with the protruding knobs on the bracket. The remote head should slide down onto the knobs, firmly and securely. The display bracket is now ready for use.

MAINTENANCE

PROTECT YOUR SCALE FROM SHOCK, OVERLOAD, LIQUID SPILLS, EXTREME TEMPERATURES, OTHER ABUSES. Preserve the finish of the scale by wiping only with damp, soft cloth and mild, nonabrasive detergent.

SPECIFICATIONS

Dimensions:

Remote Head: 5" x 3" x 1 1/4"

Platform Size: 12" x 12 1/4" x 2 1/4"

Power Source: 9 Volt battery or 110V AC adapter output: 9V DC 100 mA (included).

Model	Capacity	Resolution
320	400 lb / 180kg	0.5 lb / 0.2kg
349	400 lb / 180kg	0.2 lb / 0.05kg

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