
Health o meter®

**ELECTRONIC CHAIR SCALE
MODEL 595KL**

**FAUTEUIL DE PESEE ELECTRONIQUE
MODELE 595KL**

**BALANZA ELECTRONICA CON SILLA
MODELO 595KL**

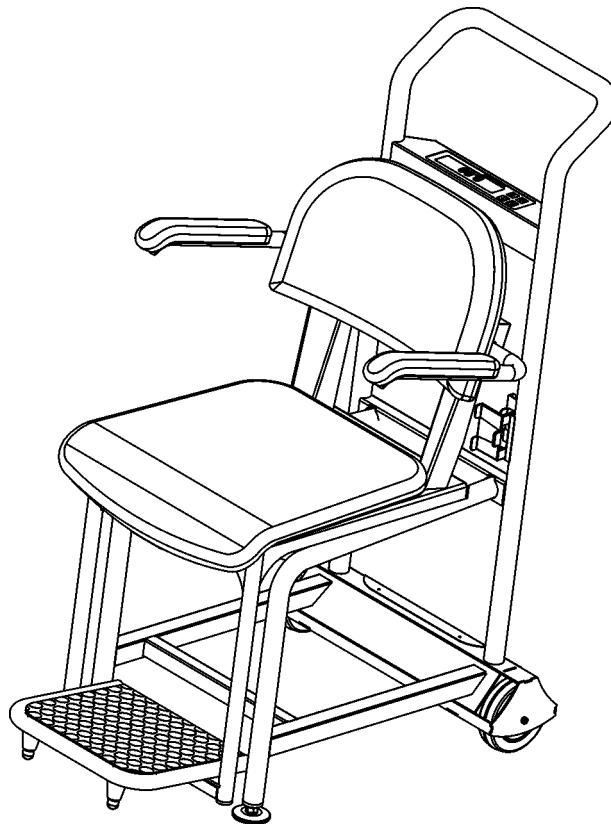OPERATION MANUAL

MODE D'EMPLOI

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ELECTRONIC CHAIR SCALE MODEL 595KL

Thank you for your purchase of this product.

Please read this manual carefully and keep it handy for ready reference.

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TABLE OF CONTENTS

| | |
|------------------------------|----|
| CAUTION AND WARNING..... | 2 |
| SPECIFICATIONS..... | 2 |
| ASSEMBLY INSTRUCTIONS | 3 |
| SET UP..... | 4 |
| OPERATION INSTRUCTIONS..... | 6 |
| MAINTENANCE..... | 6 |
| TROUBLESHOOTING..... | 7 |
| CALIBRATION PATH | 8 |
| EXPLODED VIEW OF SCALE | 9 |
| PARTS LIST | 10 |
| WARRANTY | 11 |

CAUTION AND WARNING

- To prevent injury, do not transport the scale while the chair is loaded.
- For accurate weighing the scale must be placed on a flat, stable surface.
- For accurate weighing, verify proper operation according to the procedure described in this manual before each use.
- Do not use in the presence of flammable materials.
- Operating at other voltages and frequencies than specified could damage the equipment.
- If the “LO BAT” indicator activates, for accurate weighing, replace the batteries or connect the scale to an AC power source as soon as possible.

SPECIFICATIONS

GENERAL

Health o meter’s Electronic Chair Scale Model 595KL uses highly sophisticated microprocessor technology. Each precision instrument is designed to provide accurate, reliable and repeatable weight measurements and features that make the weighing process simple, fast and convenient.

The weighing is performed by motion-sensing weighing technology, to determine the actual weight of a moving patient.

The weight can be display in pounds or in kilograms. The scale includes a padded seat with arm supports that fold up for easy entry/exit. Also included in the scale are two large wheels and a handrail for easy mobility.

The unit can be operated using its AC adapter or by 6-C cell batteries.

SCALE SPECIFICATIONS

| | | |
|--------------------------------|---|--|
| Capacity and Resolution | 600 Lb x 0.2 Lb (270 Kg x 0.1 Kg) | |
| Power Requirements | Adapter 120VAC - 60Hz 6 x C size batteries (not included) | |
| Environmental | Operating temperatures: 50°F to 95°F (10°C to 35°C) Storage temperatures: 30°F to 125°F (0°C to 50°C) Humidity: 85% | |
| Physical Dimensions | Seat Length: 17¼” (44 cm) Width: 17¾” (45 cm) | Entire Scale Length: 23¼” (59 cm) Width: 35” (89 cm) Height: 40⅞” (102 cm) Weight: 83½Lb (38Kg) |

ASSEMBLY INSTRUCTIONS

Parts List:

- (1) Chair Scale Assembly
- (2) Wheels
- (2) Plastic Feet
- (1) Arm Support

This 595KL scale has been calibrated at the factory. Remove each assembly from its carton and unwrap packing material carefully to prevent scratching the unit's parts.

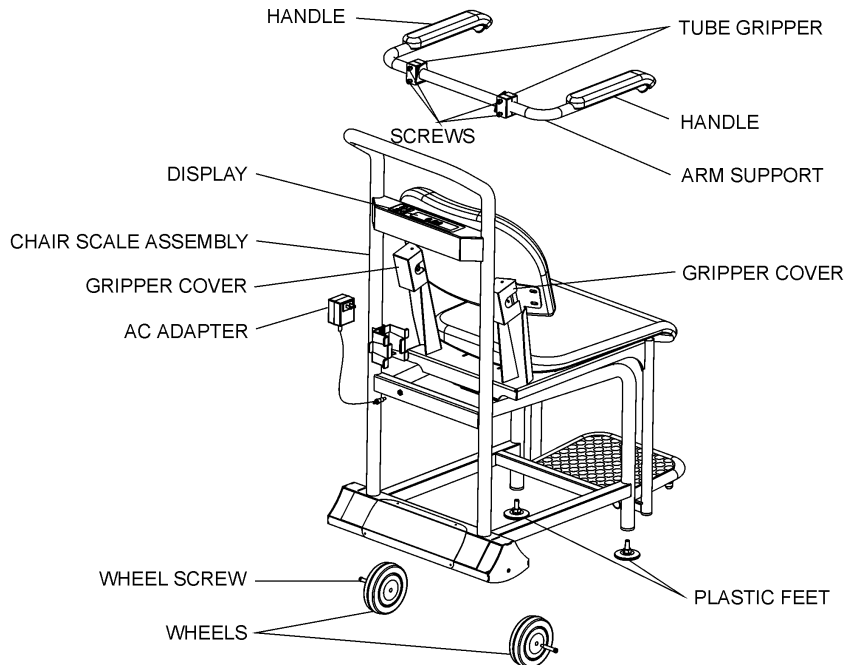
Figure 1. Assembly After Shipment

1. Place the chair scale assembly upside down on a level floor.
2. Unscrew 2 screws from both wheel supports. Attach the wheels to both sides of the chair assembly and replace the screws (use 5/32" or 4mm Allen wrench).
3. Screw the two plastic feet onto the chair assembly.
4. Turn the chair scale assembly right side up and adjust the plastic feet for equal stabilization of the chair assembly.
5. Unscrew 2 screws from each gripper cover and remove both covers (use Phillips screwdriver). Keep these screws for reinstalling in the same locations.
6. Hold the arm support with the handles upward, attach the support behind the backrest, aligning the tube grippers with the gripper covers' location. Screw 2 screws located on each tube gripper. Do not tighten the screws at this stage (use 3/16" or 5mm Allen wrench).
7. Tighten the upper screw at each gripper and then tighten the lower screws.
* Tightening of lower screws determines the freedom of motion of the arm support.
8. Reinstall both gripper covers using 4 screws removed in step 5.

The scale may be operated by an AC power source adapter or by 6-C cell batteries. The scale will automatically switch to battery operation when an AC power source is absent.

SET UP

1. Plug the scale's AC adapter into the chair scale assembly and into the power source or place batteries in the battery holder (see "Replacing Batteries" below).
2. Press the ON/OFF button to turn the scale on. The display will show "start" and then "0.0".
3. When the self-test function is complete:
 - The digital display should read "0.0".
 - "LB" should be indicated on the right side of the display. (To change the default units of the scale to kilograms (Kg), please contact our Technical Support Center at 1.800.638.3722 or 1.708.598.9100).
4. Place a weight [not to exceed 600 Lb (270 Kg)] on the scale. The digital display should blink "-----" until the weight calculation is complete.
5. Press the LBS/KG button to select the weight mode, either in kilograms or pounds.
6. Press the REWEIGH button; the digital display should read again the weight of the item on the scale.
7. Remove the weight from the scale. The digital display should continue to read the weight of the item that was on the scale.
8. Press the ZERO button; the digital display should return to zero.
9. Disconnect the power source connector from the scale, then turn the scale on using ON/OFF button. The scale should automatically switch to battery power.

NOTE: If the set up procedure failed, go to the troubleshooting instructions. If the problem is not corrected, refer to qualified service personnel.

Replacing Batteries

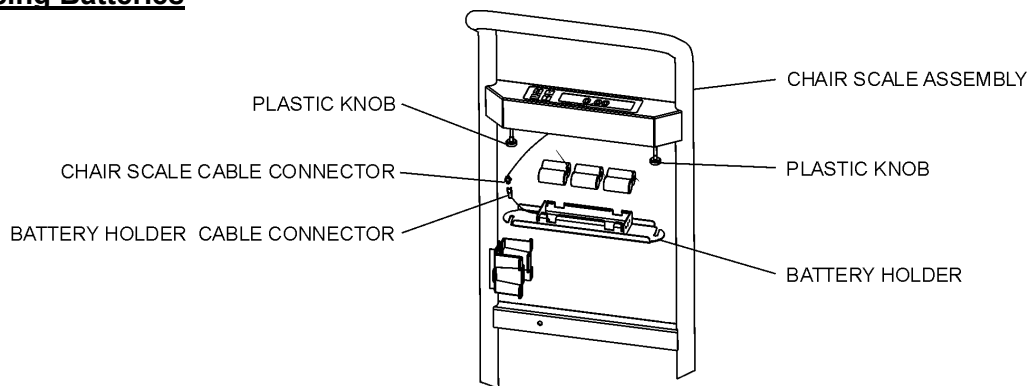
Figure 2. Replacing Batteries

1. Disconnect the scale from the AC power source.
2. Loosen two plastic knobs found underneath the display until the battery holder is released from the chair scale assembly and rests on the knobs. Carefully pull out the cable to provide access to the connector. Disconnect battery holder cable connector from the chair scale cable connector by squeezing the plastic button.
3. Raise the battery holder and simultaneously rotate it counterclockwise 1/4 turn. Carefully remove the holder from the chair scale assembly.
4. Replace the batteries with new ones.
5. Place the battery holder on the plastic knobs by rotating it 1/4 turn clockwise.
6. Plug the battery holder cable connector to the chair scale cable connector.
7. Hold the battery holder close to the chair scale assembly and tighten the two plastic knobs to secure in place. Be careful not to damage the cable.

SET UP

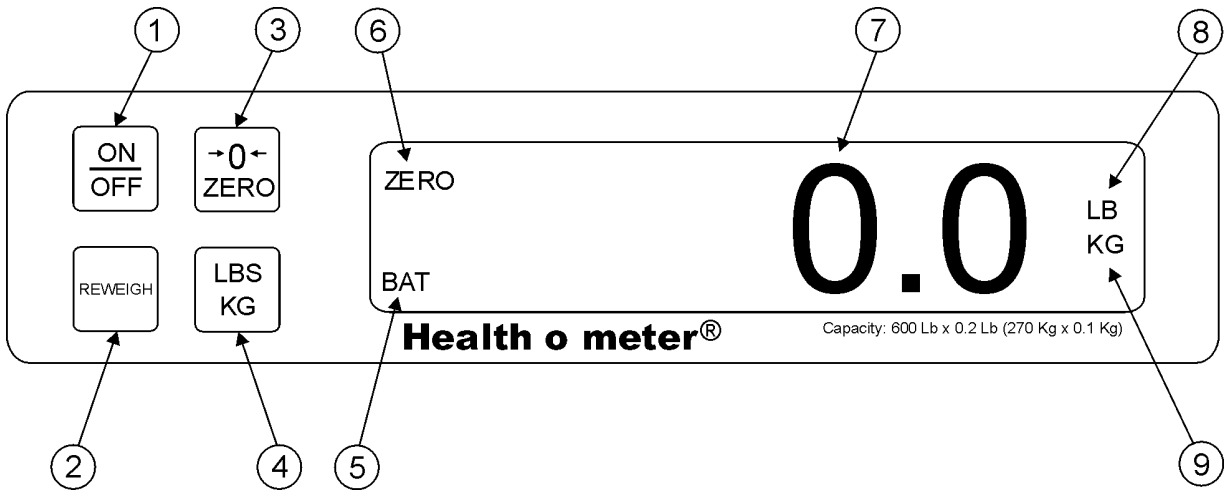
Figure 3. Control Panel

| ITEM | DESCRIPTION | FUNCTION |
|------|-----------------------------|--|
| 1 | ON/OFF button | Turns scale ON and OFF. |
| 2 | REWEIGH button | Allows repeated reweighing of person. |
| 3 | ZERO button | Zeros Digital Display prior to weighing. Enables the cancellation (tare) of weight if some object was placed on the scale. |
| 4 | LBS/KG button | Selects kilograms or pounds. |
| 5 | BAT indicator | When ON, indicates scale batteries are low. The batteries should be replaced. |
| 6 | ZERO indicator | When ON, indicates the scale is stable and ready for use. |
| 7 | Digital Display 6 Digit LCD | Displays weight in kilograms or pounds as shown by the "LB" or "KG" indicator on the right side of the display. When REWEIGH button is depressed, display will blink "-----" until patient's weight is determined. |
| 8 | LB indicator | When ON, indicates displaying weight in pounds. |
| 9 | KG indicator | When ON, indicates displaying weight in kilograms. |

OPERATION INSTRUCTIONS

1. Press the ON/OFF button to turn the scale on.
2. Wait until "0.0" appears on the digital display.
3. Sit the patient on the scale. The display will blink "-----" until the weight of the person is determined.
4. If you wish to reweigh without asking the patient to get off and to sit on the scale again, press the REWEIGH button.
5. Ask the patient to get off the scale. The digital display should continue to read the person's weight until REWEIGH or ZERO button is pressed.

NOTE: After 2 minutes of inactivity, the scale will turn itself off to conserve power. To restart the scale press the ON/OFF button.

MAINTENANCE

GENERAL

This section provides instructions for maintenance, cleaning, troubleshooting and operator replaceable parts for the Electronic Chair Scale Model 595KL. Maintenance operations other than those described in this section should be performed by qualified service personnel.

MAINTENANCE

Before first use and after periods of non-use, check the scale for proper operation and function. If the scale does not operate correctly, refer to qualified service personnel.

1. Check overall appearance of the total scale for any obvious damage, wear and tear.
2. Inspect AC adapter for cord cracking or fraying or for broken or bent prongs.

CLEANING

Proper care and cleaning is essential to ensure a long life of accurate and effective operation.

Disconnect the scale from the AC power source.

1. Clean all external surfaces with a clean damp cloth or tissue. Mild soap and water solution may be used. Dry with a clean soft cloth.
2. Do not immerse the chair scale or arm support into cleaning or other liquid solution.
3. Do not use Isopropyl Alcohol or other solutions to clean the display surface.


TROUBLESHOOTING

Refer to the following instructions to check and correct any failure before contacting service personnel.



| SYMPTOM | POSSIBLE CAUSE | CORRECTIVE ACTION |
|--|---|--|
| Scale does not turn on | <ol style="list-style-type: none"> 1. Dead Battery 2. Faulty electrical outlet 3. Bad power supply | <ol style="list-style-type: none"> 1. Replace batteries 2. Use a different outlet 3. Replace adapter |
| Questionable weight or the scale does not zero | <ol style="list-style-type: none"> 1. External object interfering with the scale 2. The display did not show "0.0" before weighing 3. Scale is not placed on a level floor 4. Scale is out of calibration | <ol style="list-style-type: none"> 1. Remove interfering object from the scale 2. Ask the patient to get off the scale, zero the scale and begin weighing process again 3. Place the scale on a level floor and begin weighing process again 4. Check weight with known weight value |
| The display shows "OL" | The load on the scale exceeds the capacity | Remove the excess weight and use the scale according to its limits |
| The display shows "UL" | The scale is in under-load condition | Make sure all seat parts and handles are installed in place and fixed properly |
| The display shows "LO BAT" | The battery is low | Replace batteries according to instructions |

CALIBRATION PATH

The modes are accessible by holding certain keys pressed after power-up. The keys must be pressed and held for at least three seconds.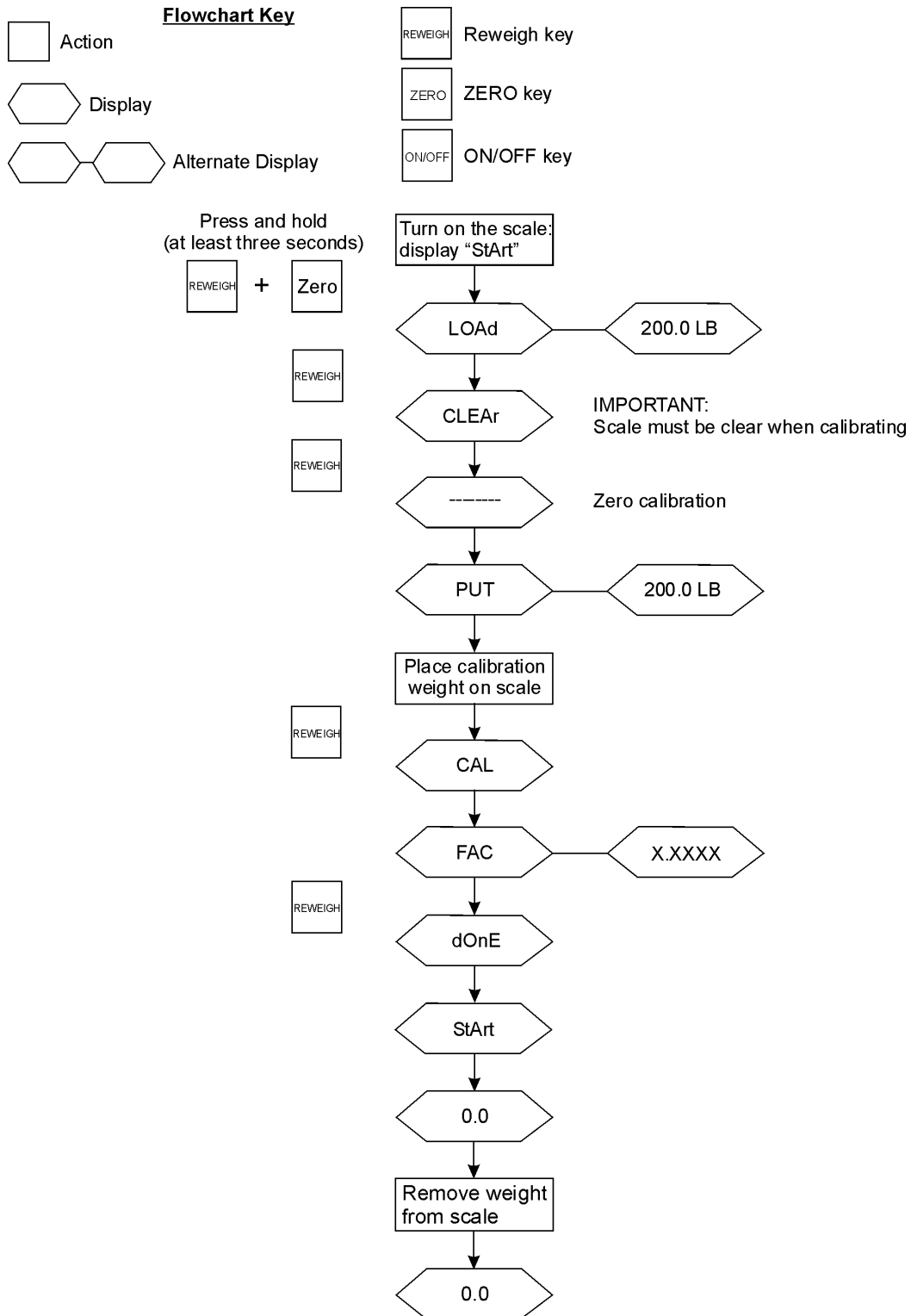

EXPLODED VIEW OF SCALE

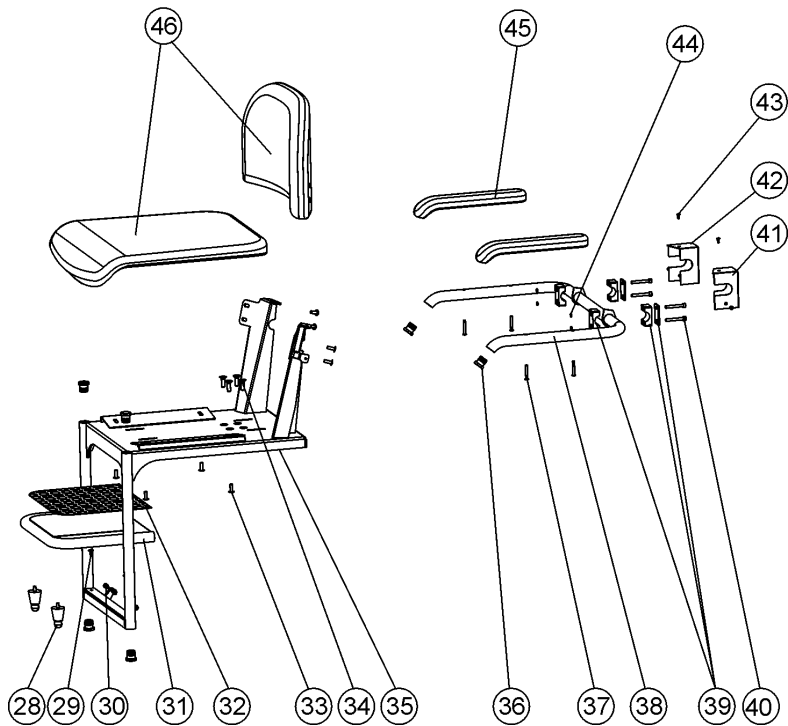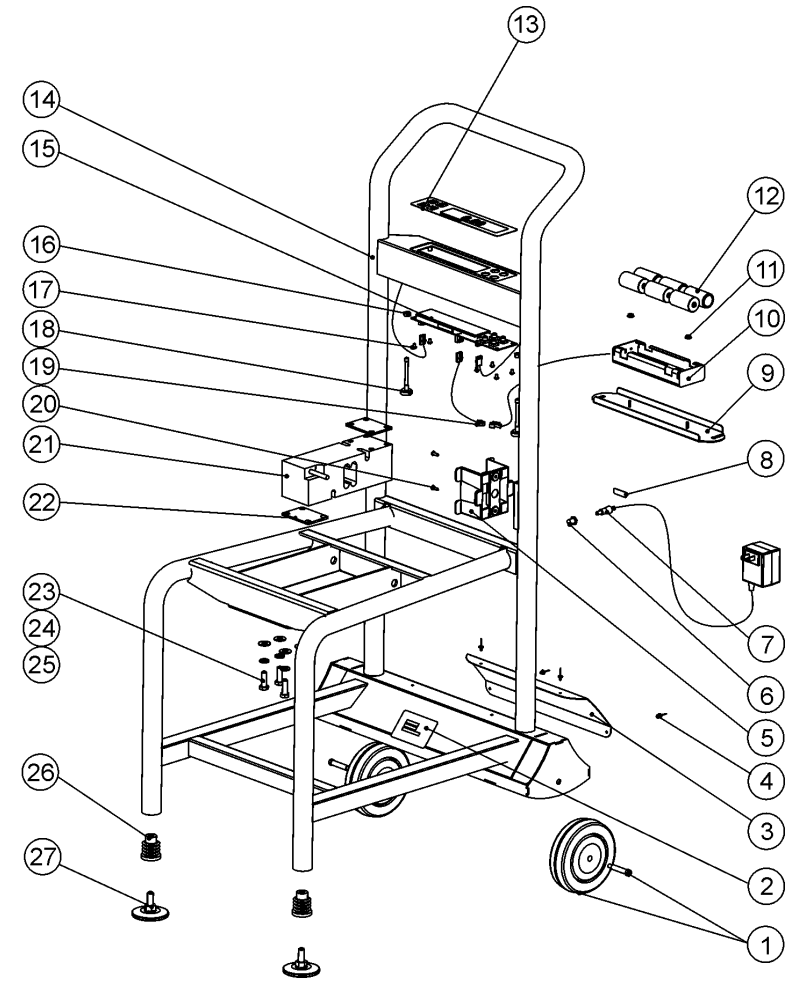PARTS LIST

| Key No. | Part No. | Description | Qty. |
|---------|-----------|--|------|
| 1 | 2266801-0 | WHEEL ASSEMBLY | 2 |
| 2 | | MODEL LABEL | 1 |
| 3 | 3817901-0 | LEG STEP PLATE | 1 |
| 4 | 400314 | POP NIT 3*6 | 4 |
| 5 | 3817001-0 | ADAPTER HOLDER | 1 |
| 6 | 2033502-0 | MAIN BOARD – ADAPTER CABLE | 1 |
| 7 | 420964 | ADAPTER UNIT 120VAC 9VDC | 1 |
| 8 | 3245101-0 | POWER SUPPLY INLET LABEL | 1 |
| 9 | 3821601-0 | DISPLAY COVER | 1 |
| 10 | 2032801-0 | BATTERY HOLDER | 1 |
| 11 | | NUT WITH LOCK WASHER NC6 | 2 |
| 12 | | C-CELL BATTERIES | 6 |
| 13 | | KEY PAD | 1 |
| 14 | | SCALE CHAIR ASSEMBLY | 1 |
| 15 | 2138803-0 | MAIN BOARD | 1 |
| 16 | | SELF LOCK NUT M6 | 4 |
| 17 | | PHIL. PAN HEAD SCREW NC6*1/4” | 6 |
| 18 | | BINDING SCREW | 2 |
| 19 | 2033301-0 | MAIN BOARD - CONN. 2 PIN CABLE | 1 |
| 20 | | SHEET METAL SCREW #8*1/2” | 2 |
| 21 | 411046 | LOAD CELL | 1 |
| 22 | 3821501-0 | SHIM FOR L.C. | 2 |
| 23 | | HEX. HEAD SCREW NC5/16*1” | 4 |
| 24 | | SPRING WASHER 5/16” | 4 |
| 25 | | WASHER 5/16” | 4 |
| 26 | 401280 | PLASTIC PIPE COVER SLEEVE NC3/8” | 2 |
| 27 | 3810002-0 | PLASTIC FOOT NC3/8” | 2 |
| 28 | 3438901-2 | RUBBER SHOCK ABSORBER | 2 |
| 29 | 401295 | RUBBER PUSH RIVET | 2 |
| 30 | 400608 | HEX. SOCKET HEAD CAP SCREW M6*16 | 2 |
| 31 | 3819601-0 | LEG SHELF | 1 |
| 32 | 3819701-0 | PVC PLATE | 1 |
| 33 | | CROWNED MUSHROOM HEAD SCREW NC1/4*3/4” | 8 |
| 34 | | SOCKET FLAT HEAD SCREW NC5/16*1” | 4 |
| 35 | 3821301-0 | SEAT BASE ASSEMBLY | 1 |
| 36 | 401289 | ROUND TUBE CAP | 6 |
| 37 | | SOCKET BUTTON HEAD SCREW M5*35 | 4 |
| 38 | 3820001-0 | ARM SUPPORT | 1 |
| 39 | 401281 | TUBE GRIPPER | 2 |
| 40 | | SOCKET HEAD CAP SCREW M6*45 | 4 |
| 41 | 3819001-0 | L. TUBE GRIPPER COVER | 1 |
| 42 | 3819101-0 | R. TUBE GRIPPER COVER | 1 |
| 43 | | PHIL. PAN HEAD SCREW M4*10 | 4 |
| 44 | 401292 | SEALING RING O.D=8.8mm | 4 |
| 45 | 401296 | HANDLE | 2 |
| 46 | 401294 | SEAT | 1 |

WARRANTY

LIMITED WARRANTY

What does the Warranty Cover?

Pelstar LLC scales are warranted from date of purchase against defects of materials or in workmanship for a period of one (1) year. If product fails to function properly, return the product, freight prepaid and properly packed to Pelstar. See "To Get Warranty Service" below for instructions. If manufacturer determines that a defect of material or in workmanship exists, customers' sole remedy will be repair or replacement of scale at no charge. Replacement will be made with a new or remanufactured product or component. If the product is no longer available, replacement may be made with a similar product of equal or greater value. All parts including repaired and replaced parts are covered only for the original warranty period.

Who is Covered?

The original purchaser of the product must have proof of purchase to receive warranty service. Pelstar dealers or retail stores selling Pelstar products do not have the right to alter, or modify or any way change the terms and conditions of this warranty.

What is Excluded?

Your warranty does not cover normal wear of parts or damage resulting from any of the following: negligent use or misuse of the product, use on improper voltage or current, use contrary to the operating instruction, abuse including tampering, damage in transit, or unauthorized repair or alternations. Further, the warranty does not cover Acts of God, such as fire, flood, hurricanes and tornadoes. This warranty gives you specific legal rights, and you may also have other rights that vary from country to country, state to state, province to province or jurisdiction to jurisdiction.

To get Warranty Service

Make sure you keep your sales receipt or document showing proof of purchase.

Call 1 (800) 815-6615 or 1 (708) 598-9100 to receive a return authorization number. Attach proof of purchase to your defective product along with your name, address, daytime telephone number and description of the problem. Carefully package the product and send with shipping and insurance prepaid to:

Pelstar LLC
Attention R/A# _____
Repair Department
7400 W. 100th Place
Bridgeview, IL 60455

If your scale is not covered by warranty, or has been damaged, an estimate of repair costs or replacement costs will be provided to you for approval prior to servicing or replacing.

Pelstar LLC
7400 West 100th Place, Bridgeview IL 60455 • 1-800-815-6615 or 1-708-598-9100
www.healthometermedical.com

Health o meter® is a registered trademark of Sunbeam Products Inc., Boca Raton, FL 33431

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Health o meter® es una marca registrada de Sunbeam Products Inc., Boca Raton, FL 33431