LIMITED WARRANTY

What does the Warranty Cover?

Pelstar LLC scales are warranted from date of purchase against defects of materials or in workmanship for a period of one (1) year. If product fails to function properly, return the product, prepaid and properly packed to Pelstar. If manufacturer determines that a defect of material or in workmanship exists, customers' sole remedy will be repair or replacement of scale at no charge. Replacement will be made with a new or remanufactured product or component. If the product is no longer available, replacement may be made with a similar product of equal or greater value. All parts including repaired and replaced parts are covered only for the original warranty period.

Who is Covered?

The original purchaser of the product must have proof of purchase to receive warranty service. Pelstar dealers or retail stores selling Pelstar products do not have the right to alter, or modify or any way change the terms and conditions of this warranty.

What is Excluded?

Your warranty does not cover normal wear of parts or damage resulting from any of the following: negligent use or misuse of the product, use on improper voltage or current, use contrary to the operating instruction, abuse including tampering, damage in transit, or unauthorized repair or alternations. Further, the warranty does not cover Acts of God, such as fire, flood, hurricanes and tornadoes. This warranty gives you specific legal rights, and you may also have other rights that vary from country to country, state to state, province to province or jurisdiction to jurisdiction.

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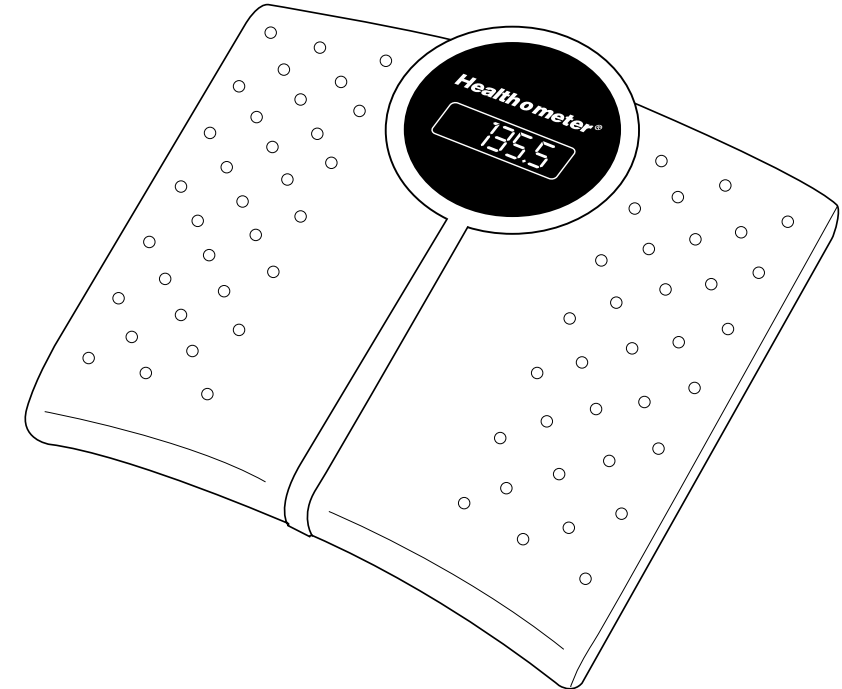
7400 West 100th Place, Bridgeview IL 60455 • 1-800-815-6615

www.healthometermedical.com

Health o meter® is a registered trademark of Sunbeam Products Inc., Boca Raton, FL 33431

Healthometer®

OPERATING INSTRUCTIONS
INSTRUCCIONES DE OPERACIÓN
MODE D'EMPLOI

DIGITAL SCALE

MODEL: 830KL

BALANZA DIGITAL

MODELO: 830KL

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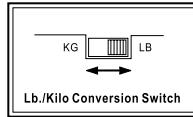
MODÈLE: 830KL

DIGITAL SCALE

MODEL: 830KL


USING THE SCALE

- 1: Set the KG/LB switch under the scale to correct setting.
- 2: Install a new 9 volt alkaline battery into the battery compartment in the scale base.
- 3: Place the scale on a hard and flat surface for best accuracy. Accuracy may be affected by very soft, deep pile carpets or rugs.
- 4: Step straight onto the scale. The display will show '00', then display weight.
- 5: Step off the scale to turn the scale off.
- 6: If weighed and a 'C' is displayed after the zero, weigh again for a more accurate reading.
- 7: Change battery when 'Lo' appears on the display.

INSTALLING BATTERY Use only 9V alkaline battery.

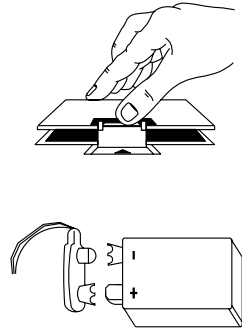- 1: Turn scale over to back side of housing and open the battery cover by squeezing and lifting tab.
- 2: Pull battery connector clip through housing to expose the clip and wires.
- 3: Place a new 9V alkaline battery into holder slots of battery cover as shown on battery cover.
- 4: Connect battery to battery compartment, make sure the '+' and '-' are positioned correctly.
- 5: Replace battery holder.

Replace the battery when the display reads 'Lo'.

MAINTENANCE

- 1: If necessary to clean the scale, wipe the outside of the scale with a soft cloth and mild detergent. Do not use abrasive cleaners. Never place the scale in water or spray with liquid.
- 2: This scale contains electronic parts. Use it carefully. Do not drop, kick or jump on it.
- 3: Do not store scale near chemicals, heaters or radiators.
- 4: Do not store scale on its side.
- 5: This scale is extremely sensitive weight device. Items such as clothing or towels placed or dropped on the scale may cause it to activate. In order to prevent run down of the battery, do not place anything on the scale.

ONE YEAR LIMITED WARRANTY See back for details.

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