## LIMITED WARRANTY

### What does the Warranty Cover?

Pelstar LLC scales are warranted from date of purchase against defects of materials or in workmanship for a period of one (1) year. If product fails to function properly, return the product, prepaid and properly packed to Pelstar. If manufacturer determines that a defect of material or in workmanship exists, customers' sole remedy will be repair or replacement of scale at no charge. Replacement will be made with a new or remanufactured product or component. If the product is no longer available, replacement may be made with a similar product of equal or greater value. All parts including repaired and replaced parts are covered only for the original warranty period.

### Who is Covered?

The original purchaser of the product must have proof of purchase to receive warranty service. Pelstar dealers or retail stores selling Pelstar products do not have the right to alter, or modify or any way change the terms and conditions of this warranty.

### What is Excluded?

Your warranty does not cover normal wear of parts or damage resulting from any of the following: negligent use or misuse of the product, use on improper voltage or current, use contrary to the operating instruction, abuse including tampering, damage in transit, or unauthorized repair or alternations. Further, the warranty does not cover Acts of God, such as fire, flood, hurricanes and tornadoes. This warranty gives you specific legal rights, and you may also have other rights that vary from country to country, state to state, province to province or jurisdiction to jurisdiction.

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**Health o meter®**

7400 West 100th Place, Bridgeview IL 60455 • 1-800-815-6615

[www.healthometermedical.com](http://www.healthometermedical.com)

Health o meter® is a registered trademark of Sunbeam Products Inc., Boca Raton, FL 33431

# Health o meter®

OPERATING INSTRUCTIONS  
INSTRUCCIONES DE OPERACIÓN  
MODE D'EMPLOI

**DIGITAL SCALE**

MODEL: 849KL

**BALANZA DIGITAL**

MODELO: 849KL

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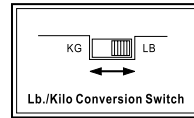
MODÈLE: 849KL

# DIGITAL SCALE

MODEL: 849KL

## USING THE SCALE

- 1: Set the KG/LB switch under the scale to correct setting.
- 2: Place the scale on a hard and flat surface for best accuracy. Accuracy may be affected by very soft, deep pile carpets or rugs.
- 3: At all times, step straight onto the scale, stand still and wait for your weight display.
- 4: Step off the scale, to turn off the scale.

## MAINTENANCE

- 1: If necessary to clean the scale, wipe the outside of the scale with a soft cloth and mild detergent. Do not use abrasive cleaners. Never place the scale in water or spray with liquid.
- 2: This scale contains electronic parts. Use it carefully. Do not drop, kick or jump on it.
- 3: Do not store scale near chemicals, heaters or radiators.
- 4: Do not store scale on its side.
- 5: This scale is extremely sensitive weight device. Items such as clothing or towels placed or dropped on the scale may cause it to activate. In order to prevent run down of the battery, do not place anything on the scale.

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