

# ***QuickMedical***

## Infantronic™ Instructions

---

1. **To convert between metric (CM) and inches (IN) or fractions** press the mode key. (IN) will be displayed to indicate inch mode and fractions. (CM) will be displayed in centimeter mode.  
**The Infantronic offers five measurement modes:**  
**1/16:** Measurements are displayed in sixteenths of an inch, and black bars just right of the fraction are used to indicate increments above the indicated value. Each bar represents one sixty-fourth of an inch.  
**1/32:** Measurements are displayed in thirty-seconds of an inch, and black bars just right of the fraction are used to indicate increments above the indicated value. Each bar represents one sixty-fourth of an inch.  
**1/64:** Measurements are displayed in sixty-fourths of an inch.  
**Decimal Inch:** Measurements are displayed in inches to the closest one-thousandth inch.  
**Centimeter:** Measurements are displayed in Centimeter to the closest one-hundredth Centimeter.
2. **To Calibrate Scale** move footboard to the head beam, with the footboard touching the head beam press the (0) button to calibrate.  
  
**To Lock the Infantronic™ so that nobody can change the current setting:**
  - Press and Hold the ON/OFF key for one or two seconds. Then while still holding the ON/OFF key press the Mode key. Then release both keys at the same time and this will lock the unit. Repeat same procedure to unlock unit. Only the ON/OFF and mode key will operate in the locked setting.
3. **Should an (Err) message appear** in the display, unplug the connector from the display, then replug. Check the calibration and calibrate if necessary. If the batteries go dead or you move the footboard to fast this may cause an (Err) Message.
4. **To change the batteries** remove the two screws on the cover of the display housing, then remove the cover. Noting the proper location shown in the battery compartment reinstall new batteries. Replace the cover and tighten the screws. **Note: Do not bend battery Clips.** Battery clips are designed to make contact after cover is screwed back in place.
5. **To Change paper** put new paper in holder. Fold paper from one corner to the other side forming a sharp corner. This will make it easier to thread paper through slot, Then pull paper through slot to the Desired length.
6. **Lubricate and clean Aluminum track** with a clean cloth and CRC 3-36 to keep clean and foot beam-moving smooth.
7. **Clean bed with none abrasive mild detergent.** Do not submerge bed into water or allow excess amounts of fluid to run down into seams of bed.
8. **\*To send data to a Computer** Push the black button in the upper right corner of the digital display box. When the computer cable is not connected to the display box the black button becomes a hold button that allows you to hold the data on the digital display. To unlock hold, press the button a second time this will bring the numbers back to the true position on the scale. When sending data to a computer, only millimeters or inches can be sent.

\*For scales with output only

***QuickMedical***

Phone 425-831-5963 Fax 425-831-6032

E-mail [sales@quickmedical.com](mailto:sales@quickmedical.com) Web Page [www.quickmedical.com](http://www.quickmedical.com)