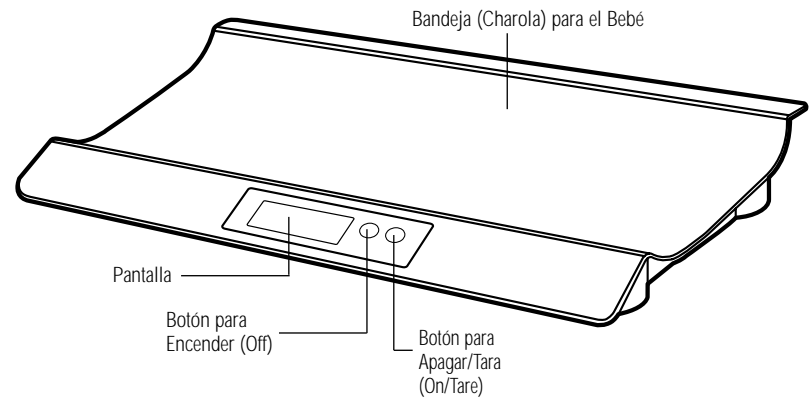
Componentes y Características

Preparaciones Antes del Uso

lb kg

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
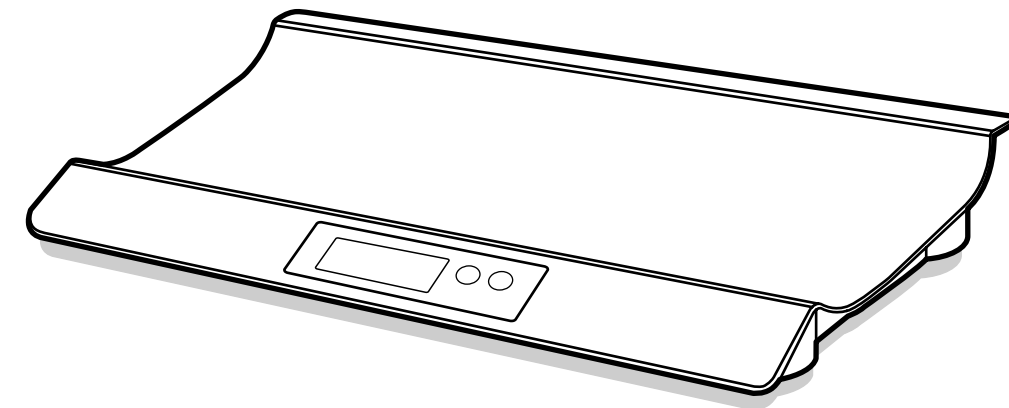
Components and Features

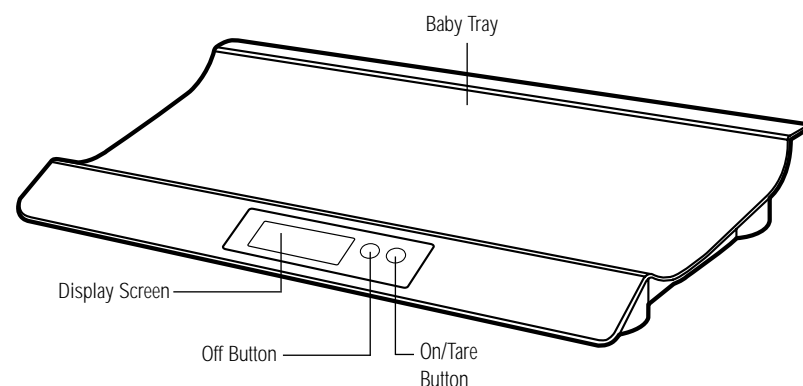
Preparations Before Use


Set scale to measure in pounds or kilograms; use the switch located on the bottom of the scale.

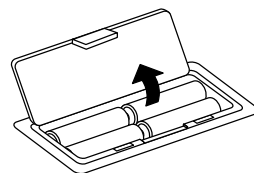Inserting the Batteries

Locate the battery cover on the bottom of the scale. Press the tab to release the cover. Insert 4 AA alkaline batteries as indicated.

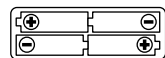Note: When you insert the battery, make sure that the negative end touches flat to the spring.


Note: For best results, use alkaline batteries. Do not mix alkaline and manganese batteries together. Avoid using rechargeable batteries.

Important Notes

- Do not get the scale wet or store in a humid place.
- The scale is a precision instrument—do not drop it, jump on or drop heavy objects onto it.
- Do not dismantle or adjust the scale.
- When pressing the On/Tare button, do not hold it down. This will cause the “Err” (error) message to appear on the display.

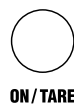
Taking Weight Measurement


Turn on the power by pressing the On/Tare button. (The display will flash all items.)

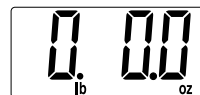
“00000” will appear on the display.

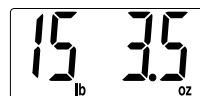Wait for the display to read “0 0.0” and place the baby on the scale (within 60 seconds).

Display will show weight.

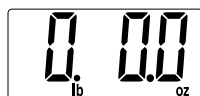After a few seconds, the display will return to “0 0.0” and turn off automatically.

Tare Function

The Tare function allows you to weigh the baby with another item (i.e. towel or blanket) without skewing the actual weight of the baby.

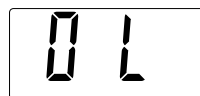
Turn on scale.

Place item (i.e. towel or blanket) on the baby tray.

Press the On/Tare button to reset scale to “0 0.0”.

Place baby on the scale.

Note: When the total weight of item and baby's weight exceeds 40 lb or 20 kg, the display shows “OL” (overload).

Replacing Batteries

If “LO” appears on the display, batteries are running down and need to be replaced. If nothing appears on the display the batteries are dead and need to be replaced.

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No. UPC

742496-858504

QuickMedical is an authorized distributor for Tanita. See the full line of tanita at <http://tanitascale.com/> Contact QuickMedical at 888-345-4858

www.tanita.com

Tanita Corporation of Japan

14-2, 1-Chome, Maeno-Cho
Itabashi-Ku, Tokyo, Japan 174-8630
Tel: 03-3968-2123
Fax: 03-3967-3766
ISO 9001 Certified

Tanita France S.A.

Villa Labrouste
68 Boulevard Bourdon
92200 Neuilly-Sur-Seine, France
Tel: 01 55 24 99 99
Fax: 01 55 24 98 68

Tanita Europe GmbH

Dresdener Strasse 25
D-71065 Sindelfingen, Germany
Tel: 07031-6189-6
Fax: 07031-6189-71

Tanita Corporation of America Inc.

2625 South Clearbrook Drive
Arlington Heights, IL 60005 USA
Tel: 847-640-9241
Fax: 847-640-9261
Toll Free: 1-877-6-TANITA

Tanita Health Equipment H.K. Ltd.

Unit 301-303 3/F Wing On Plaza
62 Mody Road, Tsimshatsui East
Kowloon, Hong Kong
Tel: 852-2838-7111
Fax: 852-2838-8667

Tanita UK Ltd.

The Barn, Philpots Close, Yiewsley,
West Drayton, Middlesex, UB7 7RY, UK
Tel: 01895-438577
Fax: 01895-438511