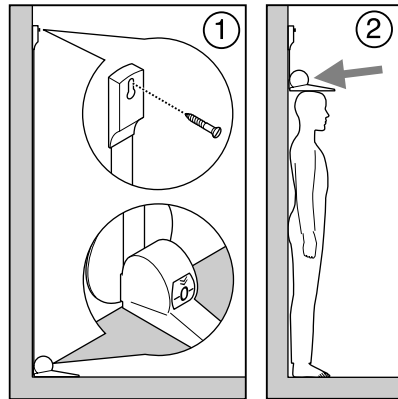
# bodymeter 206

## Herzlichen Glückwunsch

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## Congratulations

In the seca 206 bodymeter you have purchased an easy-to-use device for measuring the height of children and adults. It is designed for use in hospitals, doctors' surgeries and in-patient measuring and nursing facilities.

## Safety

Before using the new bodymeter, please take a little time to read the following safety instructions.

- Follow the operating instructions.
- Use the mounting material supplied.
- After measuring, push the bodymeter back up so that there is no risk of injury.

## Attaching the bodymeter to the wall

1. Place the bodymeter on the floor against the wall.
2. Pull the tape measure out far enough for the zero to line up exactly with the red stripe in the read-off area. Then mark the site of the

hole on the wall in the upper part of the attachment hole.

3. Attach the end of the tape measure to the wall using the screw supplied.
4. Push the bodymeter back together.

## Measuring with the bodymeter

- a. The person to be measured stands under the bodymeter.
- b. The measuring tongue is lowered onto the head.
- c. Height appears in the read-off area.

If the above-mentioned steps are followed, measurement accuracy is better than +/- 5 mm.

## Cleaning

Clean the measuring rod as required using a domestic cleaning agent or commercially-available disinfectants. Follow the manufacturer's instructions.

## Technical data

Measuring range: 0 - 220 cm | 87 ins  
 Graduations: 1 mm | 1/8 in  
 Dimensions (W x H x D): 35 x 115 x 150 mm  
 Weight: approx.140 g  
 Medical product as per Directive 93/42/EEC: Class I

## Disposal

If it is no longer possible to use the measuring device, your responsible waste disposal authority will be pleased to provide information about the measures necessary to dispose of it properly.

## Guarantee

We guarantee defects attributable to poor materials or workmanship for 24 months from the date of purchase.

The guarantee is invalidated if the device is opened by unauthorised parties.

**Important:** guarantee valid only in conjunction with proof of purchase.

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### CE Declaration of Conformity

Mechanical device for measuring height  
 Model: seca 206 bodymeter  
 Serial no.: see underside of bodymeter  
 The measuring device complies with the fundamental requirements of Directive 93/42/EEC pertaining to medical products.

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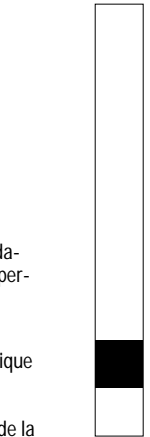
seca Vogel & Halke GmbH & Co

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Hamburg

QuickMedical is an authorized distributor for seca.  
 See the complete line of seca products at : [www.medical scales.com/seca](http://www.medical scales.com/seca)  
 Call toll free: 1-888-345-4858  
 Fax: 425-831-6032